

COOLING HEADBAND

with integrated magnets

The cooling headband combines the healing effect of cold therapy with powerful neodymium-magnets. The headband with 2 adaptable gel cushions is handy, compact and flexible and can be used wherever you want – on the forehead, ears, throat or neck. Perfect for at home and on the go!



+ Relieves pain

+ Relaxes the head

+ Cools the temples & forehead

Head

Headaches are known to almost everyone. According to experts, there are 200 different kinds and migraines and tension headache are among some of the most frequent forms. Headaches can be caused by stress, dehydration, sensitivity to weather, cold or simply just wearing the wrong glasses - the range of causes is broadly diversified.



Neck and shoulders

The main causes of neck pain are posture problems and overburdened muscles in the back, neck and shoulder area. Consistent bad posture, a cold breeze and a wrong sleeping position, frequently result in agonizing pain in the neck area which will be felt latest on the following day.



Ears

The ear is a very sensitive sensory organ, where many nerve endings are located. Even small disturbances can cause pressure on the ear and ear pain. Inflammation or infections of the ear can often be the cause for earache.



Jaw

Pain in the entire head area and in the face can even reach all the way to your teeth. Sometimes even a simple cold can cause discomfort in teeth. Misalignment of jaw or teeth can also cause pain.



Product description:

The MAGNETIX cooling headband, in the format 69 x 5.5 cm, offers soothing help for pain, swelling and small wounds. The cooling headband is very suitable for both cold and warm applications (water bath).

- With a skin-friendly cotton cover
- With 6 powerful neodymium magnets (1,200 Gauss)

Included is a cotton cover which protects your skin from the cold or heat and is soft and supple. You can easily and comfortably attach the headband to your head by means of the practical velcro fastenings.

Instructions for use:
Information for the user.

STAY COOL!

Cooling headband



Material: PVC, blue gel
Dimensions: 69 × 5,5 cm
Cover: 100% cotton
With 6 neodymium magnets (1,200 Gauss)



Application and use:

The MAGNETIX cooling headband is used for cold therapy but is also very good for heat therapy. The headband is adjustable in size and easy to use.



Applications for cold therapy*

- Migraines and headaches
- Tension
- Swelling after cosmetic surgery
- Bruises
- Weepy or swollen eyes
- Eye rings
- Toothache
- Rashes and itching
- Inflammation
- Heat complaints
- Sunburn
- Fever

Instructions for use as a cold compress:



Place the cooling headband (without cover) in the refrigerator or freezer for about **1 hour** before use. Use only with the cover. After use, remove the cover and keep it somewhere handy. You can keep the cooling headband in the freezer compartment ready for use at any time.



Applications for heat therapy*

- Tension
- Persistent headaches
- Neck pain
- Sore throat
- Chills
- Sore muscles
- Warm up prior to sports
- Promotes good blood circulation

Instructions for use as a thermal compress:



Before use, place the headband (without cover) in a water bath (up to 50° C) for about 6 minutes. Use only with the cover. After use, remove the cover and keep it somewhere handy.

Attention: Overheating may cause damage or even destroy the headband.
Not suitable for microwave ovens!

*These recommendations are not a substitute for medical treatment.



Also available:
Cooling cushion
Cooling knee pad

Warning: Do not use on children under the age of three years. Application for the treating of arterial circulatory problems, hypersensitivity to cold, acute cardiac & circulatory diseases, only after consultation with your doctor. Place the headband in the enclosed cover before use. The gel is easily biodegradable and non-toxic. Never leave children alone with

the gel pad. If a child or adult has swallowed the gel content, it is recommended to urgently consult a doctor. Damaged headbands should no longer be used. The headband should never be in contact with a hot surface. Not suitable for microwave ovens! The headband can be safely disposed of in ordinary household waste.
Attention: For external use only!

Your MAGNETIX Wellness Consultant: